

VAWC33 | HAND & WRIST ORTHOSIS

Now available directly from Frontline Medical Services

Previously on VA Denver Logistics Center (DLC) Contract

FEATURES

- Wrist Extension Control Cock Up, 9" length
- Koolflex™ material - breathable, latex-free, hypo-allergenic
- Unique hook and loop catch closure for easy donning & doffing
- Circumferential latex-free, perforated, elastic compression strap secures palmar and dorsal stays to help limit motion and secure proper fit
- Specify Right or Left hand when ordering

INDICATIONS

- Carpal tunnel syndrome · Wrist Fractures · Night Splinting
- Wrist Sprains/Strains · Post-casting

SIZING GUIDE

SIZE	WRIST CIRCUMFERENCE
XSMALL	4" - 5"
SMALL	5" - 6"
MEDIUM	6" - 7"
LARGE	7" - 8"
XLARGE	9" - 10"
2XLARGE	10" - 11"
3XLARGE	11" - 12"

CONTRAINDICATIONS

DO NOT wear if you have circulatory problems or are susceptible to dermatitis. If rash, blisters, or skin irritations occur, consult your physician and discontinue use of the support.

Made in the USA by New Options Sports



(Rev 10/18/2018)



WASHING INSTRUCTIONS: Remove palmar and dorsal splints. Close all hook and loop closures. Hand wash in cold water with mild detergent. NO chlorine bleach. Lay flat and air dry. Do NOT put in dryer.

WARRANTY: Defects in manufacturing or materials used will be covered for a period of (90) days from date of purchase. Items covered by warranty will either be replaced or repaired by the distributor. Supports must be returned CLEAN & DRY. Neither the manufacturer nor distributor will be liable for any damages or injuries arising from defective materials, workmanship, or problems covered under the Indications and Contraindications.

ORDERS: fax 720-221-0677 | orders@frontlinemedicalmil.com | phone 866-681-0444




Frontline Medical Services
Toll Free: 866-681-0444
www.frontlinemedicalmil.com



Serve proud. Live well.™


VAWC33 FITTING INSTRUCTIONS

1. Slide hand into support. The thumb placed into the smaller hole and remaining fingers in the larger hole.

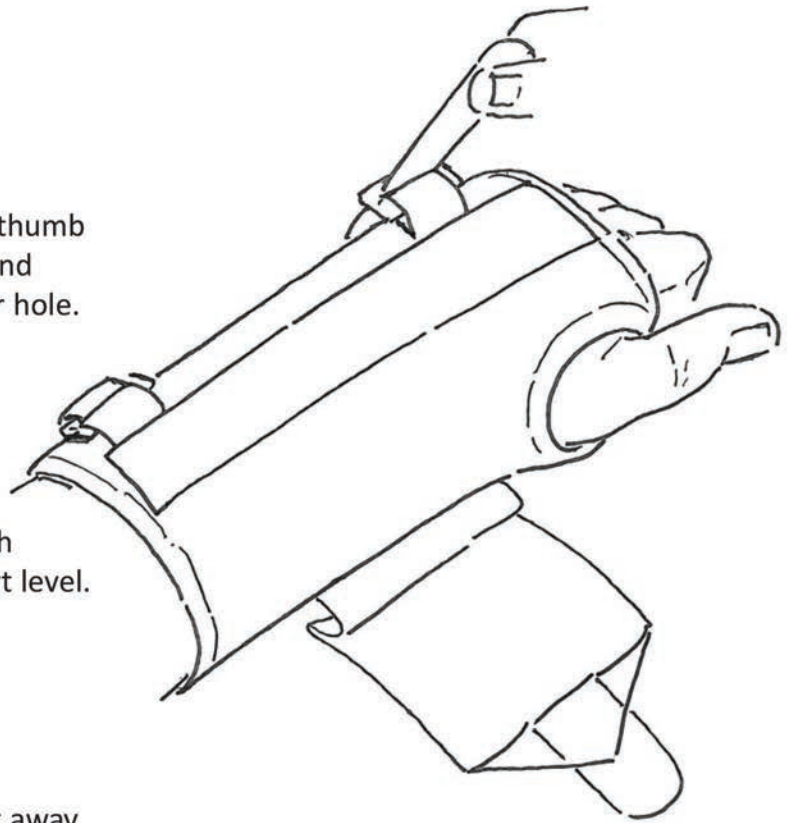

2. Thread small 1" straps through buckles and tighten to comfort level.

3. Wrap large strap around wrist away from your body and wrap all the way around support and adjust to comfort level.

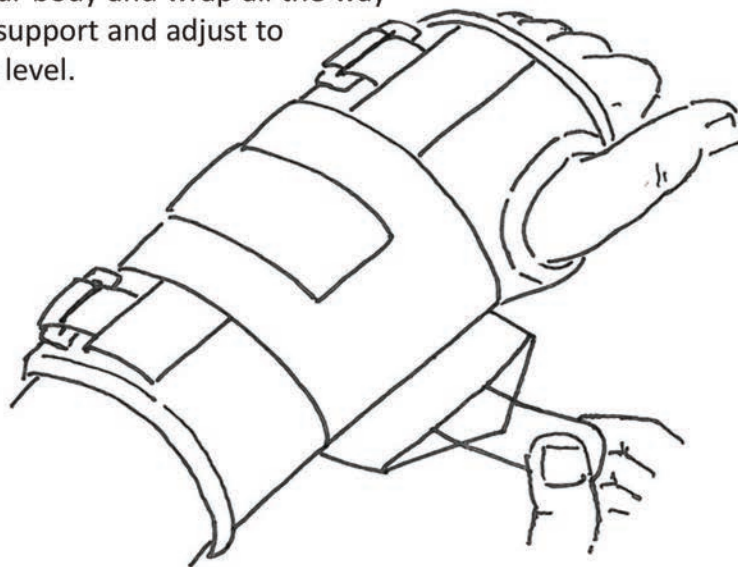

Made in the USA by New Options Sports

Frontline
MEDICAL SERVICES

Serve proud. Live well.™