

# VAKC64 | KOOLFLEX™ WRAPAROUND KNEE ORTHOSIS

## Now available directly from Frontline Medical Services

Previously on VA Denver Logistics Center (DLC) Contract

### FEATURES

- 1/8" Koolflex™ material - breathable, latex-free, hypo-allergenic
- Wrap around design
- Covered aluminum hinges
- 2" hook and loop circumferential straps
- Popliteal relief

### INDICATIONS

- Meniscus injuries
- Mild OA - Osteoarthritis
- Post-operative rehabilitation
- Medial and Lateral Collateral Ligament Instability (MCL, LCL)
- Anterior and Posterior Cruciate Ligament Instability (ACL, PCL)

### SIZING GUIDE

Measure knee circumference with leg extended.

SIZE	KNEE CENTER	6" ABOVE KNEE
XSMALL	12" - 13"	16" - 19"
SMALL	13" - 14"	17" - 20"
MEDIUM	14" - 15"	18" - 21"
LARGE	15" - 16"	19" - 22"
XLARGE	16" - 18"	20" - 24"
2XLARGE	18" - 20"	22" - 26"
3XLARGE	20" - 22"	24" - 28"
4XLARGE	22" - 24"	26" - 30"
CUSTOM	24" and Up	Provide Measurements

### CONTRAINDICATIONS

DO NOT wear if you have circulatory problems or are susceptible to dermatitis. If rash, blisters, or skin irritations occur, consult your physician and discontinue use of the support.

(Rev 10/18/2018)



(L-1810)

Made in the USA by New Options Sports

**WASHING INSTRUCTIONS:** Remove hinges from pockets. Close all hook and loop closures. Hand wash in cold water with mild detergent. NO chlorine bleach. Lay flat and air dry. Do NOT put in dryer.

**WARRANTY:** Defects in manufacturing or materials used will be covered for a period of (90) days from date of purchase. Items covered by warranty will either be replaced or repaired by the distributor. Supports must be returned CLEAN & DRY. Neither the manufacturer nor distributor will be liable for any damages or injuries arising from defective materials, workmanship, or problems covered under the Indications and Contraindications.

ORDERS: fax 720-221-0677 | [orders@frontlinemedicalmil.com](mailto:orders@frontlinemedicalmil.com) | phone 866-681-0444




Frontline Medical Services  
Toll Free: 866-681-0444  
[www.frontlinemedicalmil.com](http://www.frontlinemedicalmil.com)




Serve proud. Live well.™

# VAKC64 FITTING INSTRUCTIONS


1. Center hole on back of knee

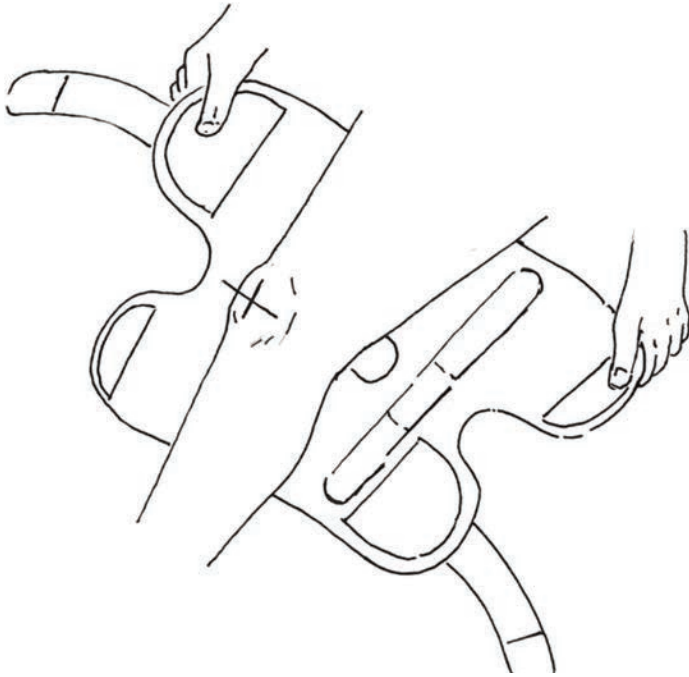


2. Wrap top portion around thigh and close Velcro then close lower portion around calf

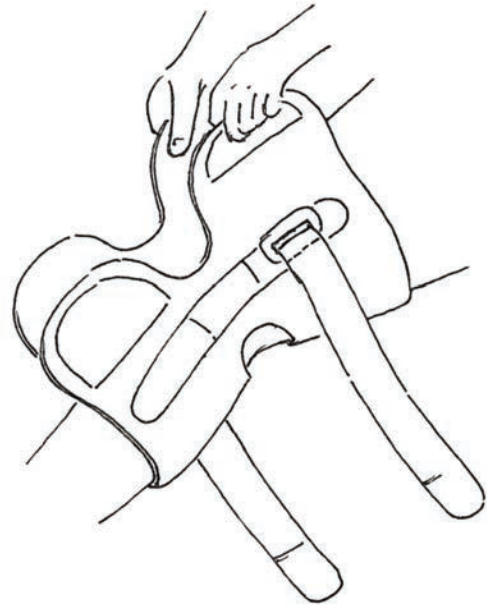

3. Wrap strap around back of leg all the way around to the front. Thread strap through buckle, adjust to comfortable compression and close Velcro

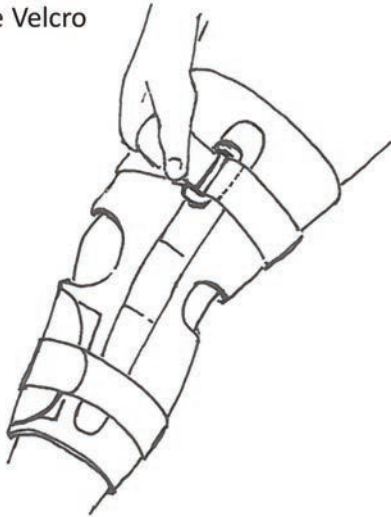

4. Support is now properly fit

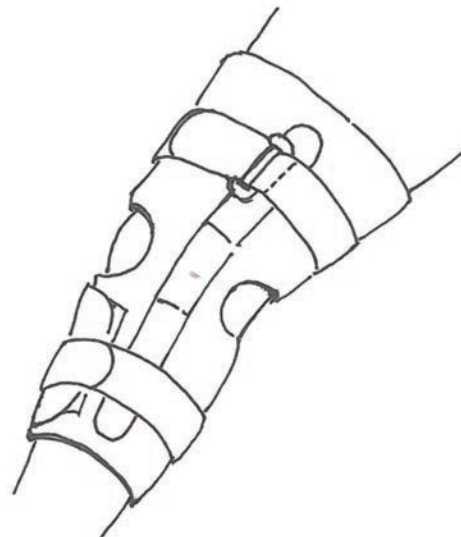

Made in the USA by New Options Sports

*Frontline*  
MEDICAL SERVICES

Serve proud. Live well.™