

VAL5N | LUMBOSACRAL SUPPORT WITH INSERT

Now available directly from Frontline Medical Services

Previously on VA Denver Logistics Center (DLC) Contract

FEATURES

- Latex-free elastic belt, 12" wide
- Two 4" wide adjustable side pulls that overlap in front to increase intra-abdominal support
- Heat moldable, semi-rigid thermoplastic insert
- Lumbar pocket to hold thermoplastic insert
- Anterior and posterior control

INDICATIONS

- Lower back pain
- Muscle spasms
- Post-operative care
- Lumbar strain
- Facet Syndrome



HEAT MOLDABLE INSERT



Made in the USA by New Options Sports

SIZING GUIDE

SIZE	WAIST CIRCUMFERENCE
XSMALL	22" - 26"
SMALL	26" - 30"
MEDIUM	30" - 34"
LARGE	34" - 38"
XLARGE	38" - 42"
2XLARGE	42" - 46"
3XLARGE	46" - 50"
4XLARGE	50" - 54"
5XLARGE	54" - 58"
6XLARGE	58" - 62"

CONTRAINDICATIONS

DO NOT wear if you have circulatory problems or are susceptible to dermatitis. If rash, blisters, or skin irritations occur, consult your physician and discontinue use of the support.

WASHING INSTRUCTIONS: Remove insert from pocket. Close all hook and loop closures. Hand wash in cold water with mild detergent. NO chlorine bleach. Lay flat and air dry. Do NOT put in dryer.

WARRANTY: Defects in manufacturing or materials used will be covered for a period of (90) days from date of purchase. Items covered by warranty will either be replaced or repaired by the distributor. Supports must be returned CLEAN & DRY. Neither the manufacturer nor distributor will be liable for any damages or injuries arising from defective materials, workmanship, or problems covered under the Indications and Contraindications.

(Rev 10/18/2018)

ORDERS: fax 720-221-0677 | orders@frontlinemedicalmil.com | phone 866-681-0444




Frontline Medical Services
Toll Free: 866-681-0444
www.frontlinemedicalmil.com




Serve proud. Live well.™

VAL5N FITTING INSTRUCTIONS

1. Place shaped insert into pocket.

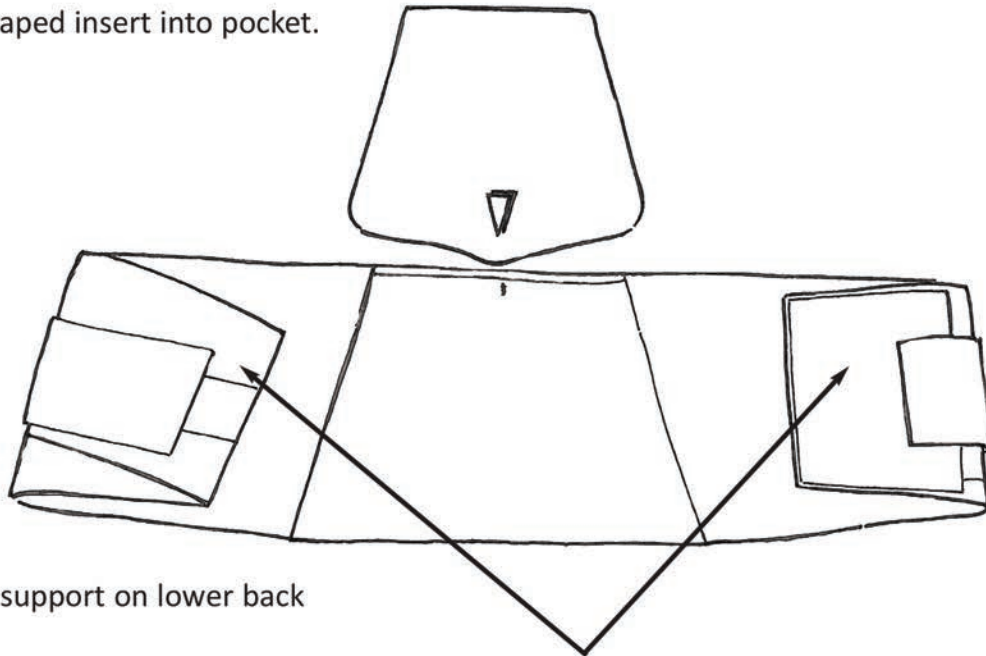

2. Center support on lower back
3. Wrap support around body and close front Velcro panels. Support should be snug, but not constrict breathing
4. Close 4" elastic side pulls and secure.

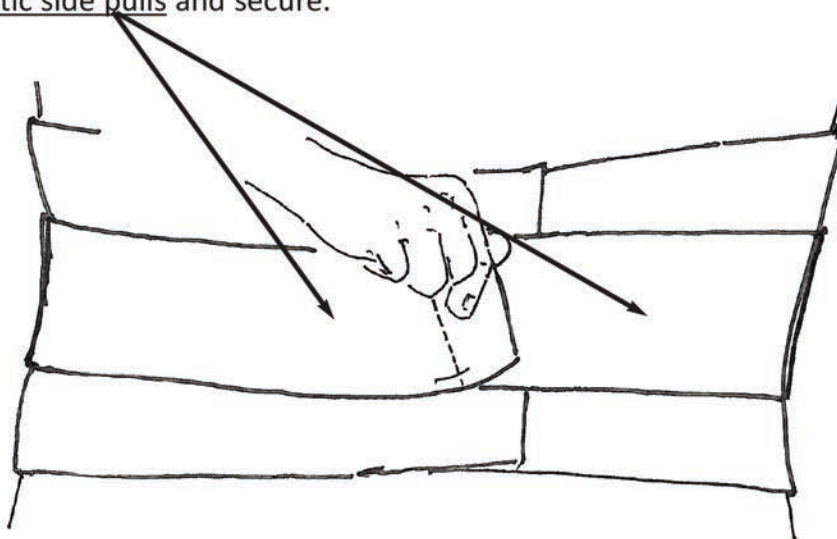

5. Support is now properly fit



New Options Sports



Made in the USA by New Options Sports

Frontline
MEDICAL SERVICES

Serve proud. Live well.™