

Now Available Directly from Frontline Medical Services

Previously on DLC Contract No: 36C79120D0009

Orders may be placed via email, fax or phone.

Slip-on Knee Sleeve with Patella Stabilizing Pad

FEATURES

- Slip-on knee sleeve
- Open patella for patella femoral support and compression
- 2" Hook and loop circumferential straps
- 1/8" Koolflex™ breathable hypo-allergenic material
- 1/4" Removable, adjustable patella stabilizing pad
- Flexible medial and lateral stays
- 12" Length
- Fits right or left knee

INDICATIONS

- Sprains and strains
- Arthritis
- Tendonitis
- Chondromalacia
- Patella tracking abnormalities
- Post-operative rehabilitation
- Sports or work related injuries

CONTRAINDICATIONS

DO NOT wear if you have circulatory problems or are susceptible to dermatitis. If rash, blisters, or skin irritations occur, consult your physician and discontinue use of the support.



New Options Sports



Frontline Medical Services
orders@frontlinemedicalmil.com
fax orders: 720-221-0677
phone: 866-681-0444
www.frontlinemedicalmil.com



Serve proud. Live well.™

VAKC19 | KOOLFLEX™ SLIP-ON KNEE ORTHOSIS

FITTING INSTRUCTIONS

STEP 1: Slide support onto the front of leg, centering patella opening over the knee

STEP 2: Thread the straps through the buckles and tighten, securing the hook closures to the straps

STEP 3: Support is now properly fit

SIZING GUIDE


Measure knee circumference with leg extended

Size	Knee Center	6" Above Knee
XSMALL	12" - 13"	16" - 19"
SMALL	13" - 14"	17" - 20"
MEDIUM	14" - 15"	18" - 21"
LARGE	15" - 16"	19" - 22"
XLARGE	16" - 18"	20" - 24"
2XLARGE	18" - 20"	22" - 26"
3XLARGE	20" - 22"	24" - 28"
4XLARGE	22" - 24"	26" - 30"

HCPC: A4467

WASHING INSTRUCTIONS: Hand wash in cold water with mild detergent. NO chlorine bleach. Lay flat and air dry. Do NOT put in dryer.

WARRANTY: Defects in manufacturing or materials used will be covered for a period of (90) days from date of purchase. Items covered by warranty will either be replaced or repaired by the distributor. Supports must be returned CLEAN & DRY. Neither the manufacturer nor distributor will be liable for any damages or injuries arising from defective materials, workmanship, or problems covered under the Indications and Contraindications.

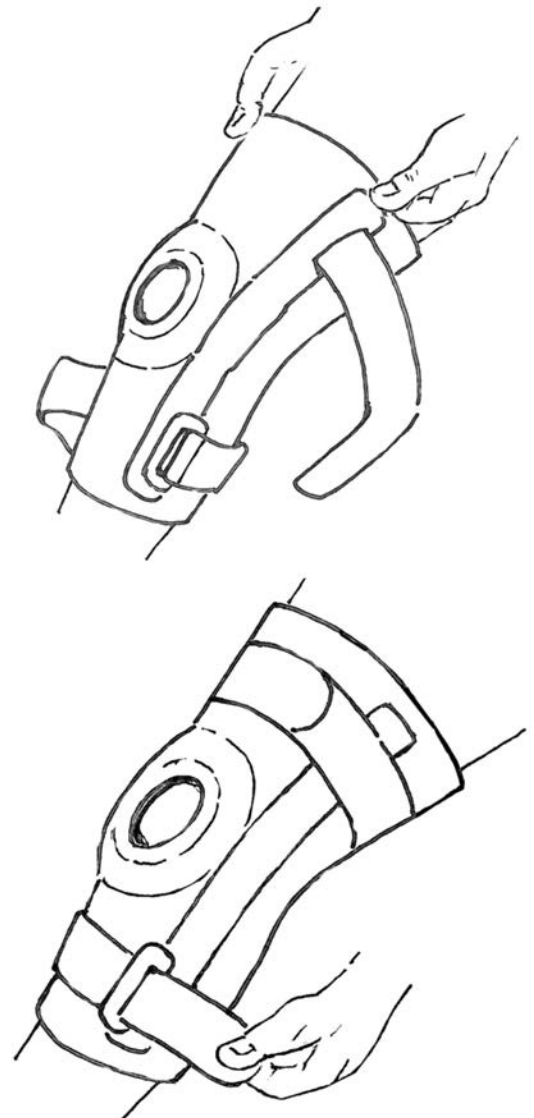

New Options Sports

Made in the USA by New Options Sports



Frontline Medical Services
orders@frontlinemedicalmil.com
fax orders: 720-221-0677
phone: 866-681-0444
www.frontlinemedicalmil.com



Serve proud. Live well.™