

VAKC46 | KOOLFLEX™ SLIP-ON KNEE ORTHOSIS

Now Available Directly from Frontline Medical Services

Previously on DLC Contract No: 36C79120D0009

Orders may be placed via email, fax or phone.

Slip-on Knee Sleeve with Positive Control Distal Strap

FEATURES

- Koolflex™ breathable hypo-allergenic material
- Lower strap guide for positive positioning and control of patella tracking
- Open patella for positioning and comfort
- Inverted Horseshoe buttress for patella support
- Hook and loop adjustable circumferential straps
- Medial and lateral spiral stays
- 12" Length
- Fits right or left knee

INDICATIONS

- Patellofemoral Pain Syndrome
- Arthritis and patella tendonitis
- Medial/lateral collateral ligament instability
- Mild to moderate Anterior and Posterior Cruciate Ligament Instability (ACL, PCL)
- Patella subluxation
- Osgood-Schlatter Disease

CONTRAINDICATIONS

DO NOT wear if you have circulatory problems or are susceptible to dermatitis. If rash, blisters, or skin irritations occur, consult your physician and discontinue use of the support.



New Options Sports



Frontline Medical Services
orders@frontlinemedicalmil.com
fax orders: 720-221-0677
phone: 866-681-0444
www.frontlinemedicalmil.com



Serve proud. Live well.™

VAKC46 | KOOLFLEX™ SLIP-ON KNEE ORTHOSIS

FITTING INSTRUCTIONS

STEP 1: Slide support onto front of leg, centering patella opening over the knee

STEP 2: Thread upper strap through the buckle and tighten, securing the hook closure to the straps

STEP 3: Thread bottom strap through slot in front of patella buttress, thread strap through buckle, adjust to comfortable compression and secure hook and loop closure to the strap

SIZING GUIDE


Measure knee circumference with leg extended

Size	Knee Center	6" Above Knee
XSMALL	12" - 13"	16" - 19"
SMALL	13" - 14"	17" - 20"
MEDIUM	14" - 15"	18" - 21"
LARGE	15" - 16"	19" - 22"
XLARGE	16" - 18"	20" - 24"
2XLARGE	18" - 20"	22" - 26"
3XLARGE	20" - 22"	24" - 28"
4XLARGE	22" - 24"	26" - 30"

HCPC: A4467

WASHING INSTRUCTIONS: Hand wash in cold water with mild detergent. NO chlorine bleach. Lay flat and air dry. Do NOT put in dryer.

WARRANTY: Defects in manufacturing or materials used will be covered for a period of (90) days from date of purchase. Items covered by warranty will either be replaced or repaired by the distributor. Supports must be returned CLEAN & DRY. Neither the manufacturer nor distributor will be liable for any damages or injuries arising from defective materials, workmanship, or problems covered under the Indications and Contraindications.

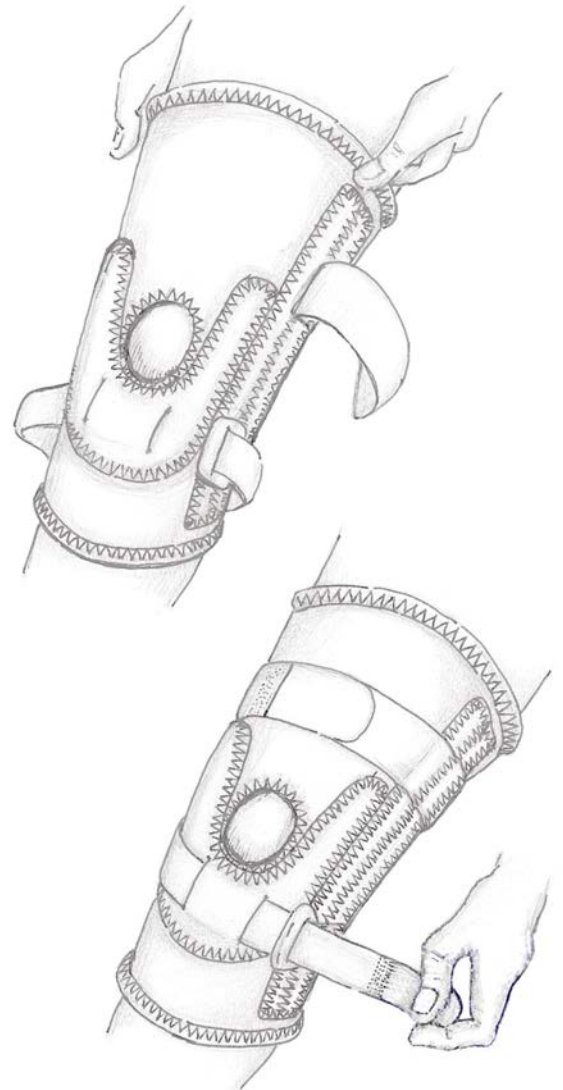

New Options Sports

Made in the USA by New Options Sports



Frontline Medical Services
orders@frontlinemedicalmil.com
fax orders: 720-221-0677
phone: 866-681-0444
www.frontlinemedicalmil.com



Serve proud. Live well.™