

VAKC9-OS | KOOLFLEX™ SLIP-ON KNEE ORTHOSIS

Now Available Directly from Frontline Medical Services

Previously on DLC Contract No: 36C79120D0009

Orders may be placed via email, fax or phone.

Slip-on Knee Sleeve with Open Patella and Spiral Stays

FEATURES

- Koolflex™ breathable hypo-allergenic material
- Open patella for comfort and support
- Medial and lateral spiral stays with dipped caps
- 12" Length
- Fits right or left knee

INDICATIONS

- Sprains and strains
- Arthritis
- Patella tracking abnormalities
- Post-operative rehabilitation
- Tendonitis
- Edema control
- Sports or work related injuries
- Chondromalacia

CONTRAINDICATIONS

DO NOT wear if you have circulatory problems or are susceptible to dermatitis. If rash, blisters, or skin irritations occur, consult your physician and discontinue use of the support.



Frontline Medical Services
orders@frontlinemedicalmil.com
fax orders: 720-221-0677
phone: 866-681-0444
www.frontlinemedicalmil.com



Serve proud. Live well.™

VAKC9-OS | KOOLFLEX™ SLIP-ON KNEE ORTHOSIS

FITTING INSTRUCTIONS

STEP 1: Slide support onto front of leg, centering patella opening over the knee


STEP 2: Pull support up using thumb and forefinger until opening is centered on knee cap

SIZING GUIDE

Measure knee circumference with leg extended

Size	Knee Center	6" Above Knee
XSMALL	12" - 13"	16" - 19"
SMALL	13" - 14"	17" - 20"
MEDIUM	14" - 15"	18" - 21"
LARGE	15" - 16"	19" - 22"
XLARGE	16" - 18"	20" - 24"
2XLARGE	18" - 20"	22" - 26"
3XLARGE	20" - 22"	24" - 28"
4XLARGE	22" - 24"	26" - 30"

HCPC: A4467

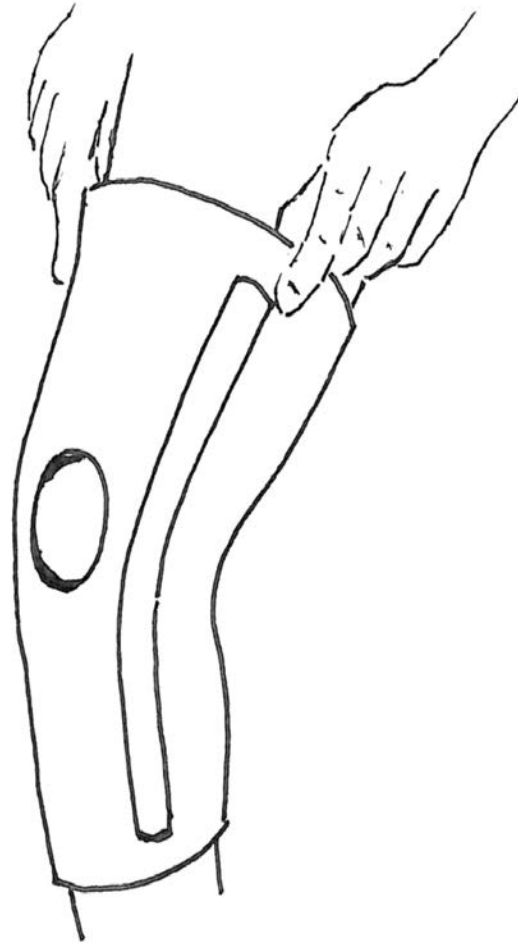

WASHING INSTRUCTIONS: Hand wash in cold water with mild detergent. NO chlorine bleach. Lay flat and air dry. Do NOT put in dryer.

WARRANTY: Defects in manufacturing or materials used will be covered for a period of (90) days from date of purchase. Items covered by warranty will either be replaced or repaired by the distributor. Supports must be returned CLEAN & DRY. Neither the manufacturer nor distributor will be liable for any damages or injuries arising from defective materials, workmanship, or problems covered under the Indications and Contraindications.



Frontline Medical Services
orders@frontlinemedicalmil.com
fax orders: 720-221-0677
phone: 866-681-0444
www.frontlinemedicalmil.com



Serve proud. Live well.™