

VALC10 | LUMBOSACRAL CORSET ORTHOSIS

Now Available Directly from Frontline Medical Services

Previously on DLC Contract No: 36C79120D0009

Orders may be placed via email, fax or phone.

Lumbosacral Corset Orthosis

FEATURES

- Breathable, hypo-allergenic posterior panel
- Latex-free elastic anterior panels
- Hook and loop Velcro closure
- Four removable pre-contoured rigid steel posterior stays for intra-abdominal support
- Two reinforced circumferential straps
- Heavy duty reinforced loops
- Length extends from sacrococcygeal junction to T9 vertebrae

INDICATIONS

- Lower back pain
- Post-operative care
- Facet Syndrome
- Muscle spasms
- Lumbar disc injury

CONTRAINDICATIONS

DO NOT wear if you have circulatory problems or are susceptible to dermatitis. If rash, blisters, or skin irritations occur, consult your physician and discontinue use of the support.



New Options Sports



Frontline Medical Services
orders@frontlinemedicalmil.com
fax orders: 720-221-0677
phone: 866-681-0444
www.frontlinemedicalmil.com



Serve proud. Live well.™

VALC10 | LUMBOSACRAL CORSET ORTHOSIS

FITTING INSTRUCTIONS

STEP 1: Center corset on lower back

STEP 2: Wrap support around front of body and secure hook closure

STEP 3: To adjust front straps longer or shorter, pull the front strap through the slip buckle located on the right side of the corset

STEP 4: Thread straps through buckles and secure to desired comfort

SIZING GUIDE




Measure around the waist

Size	Waist Circumference
XSMALL	20" - 24"
SMALL	24" - 30"
MEDIUM	30" - 36"
LARGE	36" - 42"
XLARGE	42" - 48"
2XLARGE	48" - 54"
3XLARGE	54" - 62"

HPC: L0630

WASHING INSTRUCTIONS: Hand wash in cold water with mild detergent. NO chlorine bleach. Lay flat and air dry. Do NOT put in dryer.

WARRANTY: Defects in manufacturing or materials used will be covered for a period of (90) days from date of purchase. Items covered by warranty will either be replaced or repaired by the distributor. Supports must be returned CLEAN & DRY. Neither the manufacturer nor distributor will be liable for any damages or injuries arising from defective materials, workmanship, or problems covered under the Indications and Contraindications.

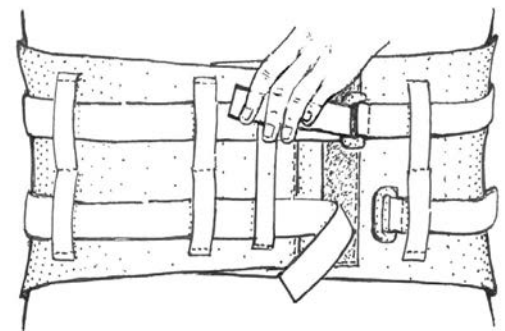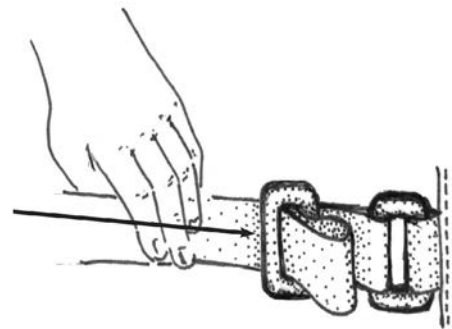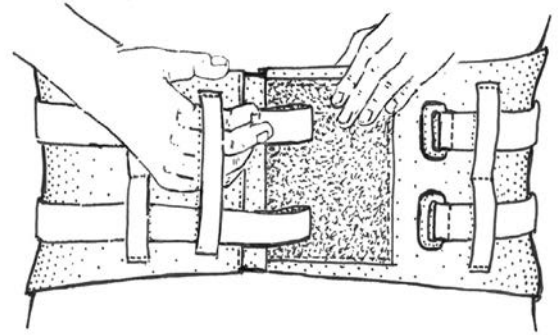

New Options Sports

Made in the USA by New Options Sports



Frontline Medical Services
 orders@frontlinemedicalmil.com
 fax orders: 720-221-0677
 phone: 866-681-0444
 www.frontlinemedicalmil.com



Serve proud. Live well.™